






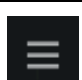






## Cerner Image Viewer Basic Function Guide

Cerner Image Viewer is a web accessible software application that displays digital imaging as reference information to support the needs of clinicians, diagnosticians, and technologists. Image Viewer can be used for reference during patient consultations.

BUTTON	NAME & ACTION
	<b>Pan (P):</b> Allows you move the image in the viewport.
	<b>Zoom (Z):</b> Allows you to adjust the size of the image. Right click and hold.
	<b>Window Level (W):</b> Interactively adjust the contrast and brightness. Left click and hold.
	<b>Reset Image (CTRL + R):</b> Resets image to original display.
	<b>Orientation Setting:</b> Allows you to change the orientation of the image.
	<b>2x2 Series Layout:</b> Displays the study in a 2x2 layout.
	<b>Cine (SPACEBAR):</b> Displays the active series in an animated or cine display.
	<b>Show Menu:</b> Toggles the left side menu for viewer version and help wiki link.
	<b>Study List:</b> Toggles the left side list of opened studies with thumbnails.
	<b>Toolbar:</b> Toggles the toolbar. Right click on image to open shortcut menu options.
	<b>Imaging History:</b> Toggles the patient's historical study list on the right side.
	Study list filter (determined by date)